



Quick Set

1. Press the Start / Stop button for Output 1 or 2.

2. Press Mode button to select waveform.

Presets: P01 Acute pain

- P02 Chronic pain
- P03 Edema 1 (1-30 Hz)
- P04 Edema 2 (1-10Hz)
- P05 Nerve Block

3. Press Select button to change the Time, Frequency, Pulse Width or Cycles.

If the LED is not illuminated under a selection it is not modifiable. Use the Up / Down buttons to increase or decrease.

4. Press the Start / Stop button to save.

5. Use the Up / Down buttons to increase or decrease intensity.

6. Use other output for second patient or different body region and repeat from Step 1.



Hill Therapeutics

A Division of Hill Laboratories

1-877-445-5020

www.HillTherapy.com

3 N. Bacton Hill Road, Frazer, PA 19355