



# How to replace the check valve on the Air-Flex

INS 126

Approved by: \_\_\_\_\_

Date \_\_\_\_\_

1. Unplug table.
2. At the foot end of the table, remove the screws that hold the plastic table skirting for the back side of the skirt. This will allow you to “flap” the skirt open at the back of the table and access the inner workings.
3. Locate the check valve in the hose assembly as shown (some models may not have the red and green collars but silver instead).
4. Remove the old valve by depressing the plastic collar around the hose (see photo 2) and then pulling out on the hose at the same time. Do this on both sides of the old check valve to remove.
5. Insert the new valve as shown in photo 1 (with red at the top and green at the bottom). NOTE: It is not necessary to depress the collar when inserting the hoses.
6. Replace skirt.

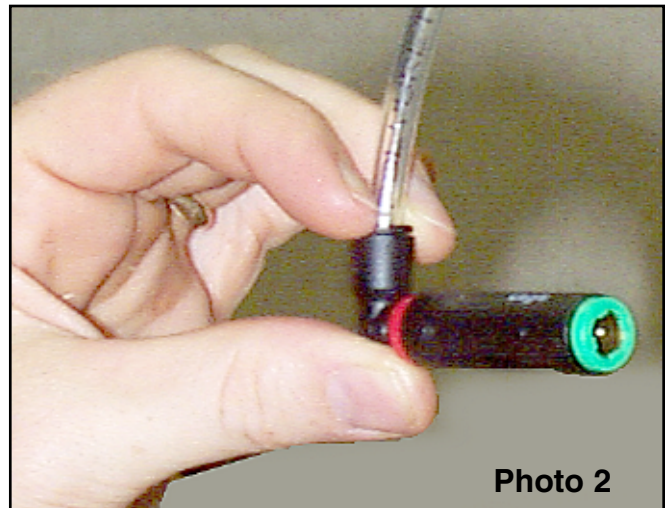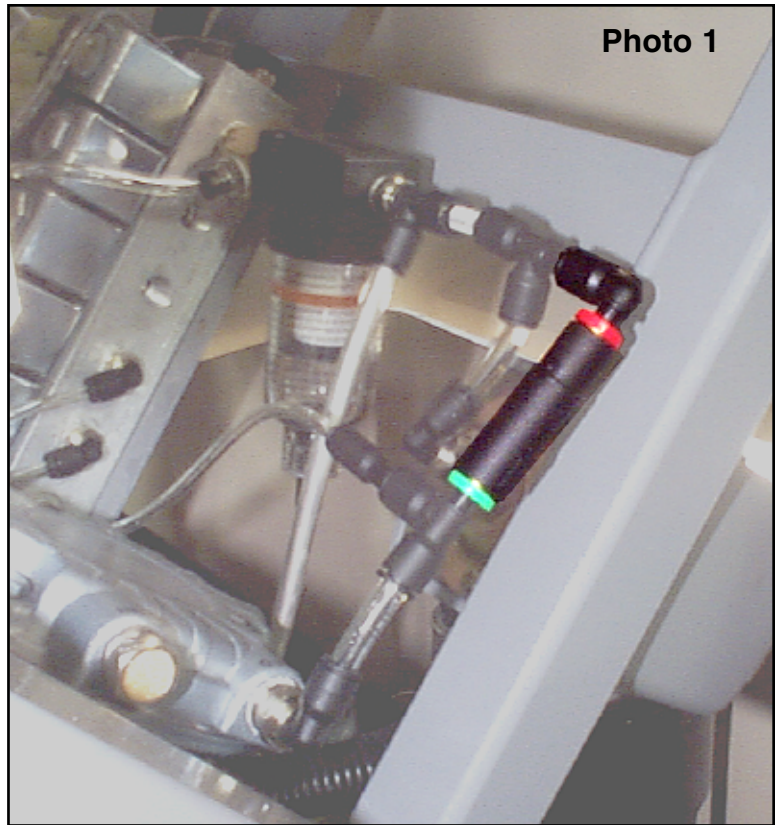

  
Hill Laboratories  
COMPANY