

Hill IFC

NEXT GENERATION



Quick Start Guide

To turn on the IFC

- Turn switch to “ON” in the back of the unit
- Attach the electrode pads to the patient
- Hand Patient Shutoff Switch to the patient
- Select Quadpolar by tapping IFC. Use arrows to select from P01 - P06. When flashing, tap to select.

IFC Presets:

- P01 Acute pain
 - P02 Chronic pain
 - P03 Edema 1 (1-30 Hz)
 - P04 Edema 2 (1-10Hz)
 - P05 Nerve Block
- Touch Start/Stop
 - Time and intensity will be blinking
 - Time is set at 10 minutes. To change, tap “Time” and use the arrows: then tap “Time” again to lock in.
 - Intensity will blink. Tap “Intensity” and use the arrows to increase intensity to patient’s comfort. Tap “Intensity” again to lock in.
 - Time and intensity can be changed while the unit is running.

To stop the treatment

- Hold the Start/Stop button for 2 to 3 seconds

To use both outputs

- Once Output 1 has been set, press and hold Output 2
- Repeat treatment selection as you did with Output 1. You may choose a completely different treatment mode and time.
- Once both modes are running, you may toggle between modes by pressing the desired output button. Press Output 1-- time and intensity will be shown, then press Output 2 to see its settings.
- To stop both outputs, you must stop one at a time. Hold the Start/Stop for 2 to 3 seconds to stop the output that is presently selected. Now select the other output and hold the Start/Stop button again for 2 to 3 seconds.

If the unit is turned off using the patient switch, both outputs will be stopped at the same time.