



Quick Set



- 1. Press the Start / Stop button for Output 1 or 2.**
- 2. Press Interferential, TENS or Galvanic buttons to select waveform.**

Presets for Interferential:

- P01 Acute pain
- P02 Chronic pain
- P03 Edema 1 (1-30 Hz)
- P04 Edema 2 (1-10Hz)
- P05 Nerve Block

- 3. Press Select button to change the Time, Frequency, Pulse Width or Cycles.**

If the LED is not illuminated under a selection it is not modifiable. Use the Up / Down buttons to increase or decrease.

- 4. Press the Start / Stop button to save.**
- 5. Use the Up / Down buttons to increase or decrease intensity.**
- 6. Use other output for second patient or different body region and repeat from Step 1.**



Hill Therapeutics

A Division of Hill Laboratories

1-877-445-5020 • www.HillTherapy.com

3 N. Bacton Hill Road, Frazer, PA 19355